

FROM DIAGNOSIS TO ORTHOSIS- ALL IN YOUR OFFICE.

The P-POD™ is a foam-in-place cranial orthosis designed to apply static or gentle pressure to an infant's head to improve the head shape. The foam insert is custom-molded to the baby's head through a simple process that can be done in less than 30 minutes.

FEATURES AND BENEFITS

- Helmet molding happens at point of care—no visits to specialists or extra appointments necessary
- Optimum helmet fit and alignment without scanning
- Non-irritating, biocompatible foam avoids creating pressure points
- Requires less adjustment than standard helmets

HCPCS Code: S1040

THE P-POD KIT INCLUDES



SCHEDULE A VIRTUAL DEMO TODAY
866.877.PPOD ppodpediatrics.com/demo

HOW IT WORKS:

- 

Use putty to fill in the flattened areas on the baby's head
- 

Place P-POD helmet on baby, securing straps
- 

Mix the foaming solution in its packet
- 

Attach the foam pouch to the bladder inside the helmet
- 

Roll up pouch to fill helmet bladder with foam solution
- 

After the foam sets, trim the foam and your P-POD is ready

**IS THE P-POD THE RIGHT FIT
FOR YOUR PRACTICE?**

SCHEDULE A VIRTUAL DEMO TODAY

866.877.PPOD ppodpediatrics.com/demo

